

NanoActive Technology

The new Trio Pearls® incorporate a superabsorbent gelling polymer and nano-engineered micro-mesoporous ingredients that actively adsorb odorous organic molecules rather than just trapping them in the gel.

Figure 1. Porosity


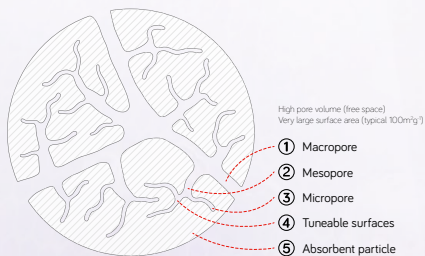


Figure 2. Adsorption

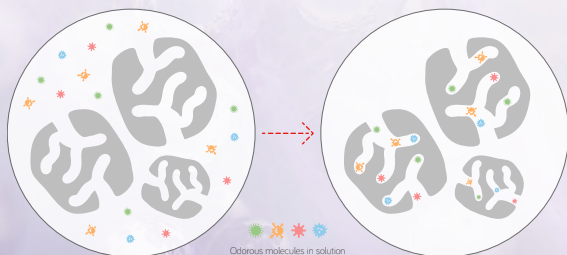

Available now
Ordering information

100 sachets

REF TR205



Try it **FREE** now!

For a **FREE** sample please contact:

Blank area for contact information.

trio ostomy care™



trio ostomy care™

TRIO HEALTHCARE LTD . MISSENDEN ABBEY . LONDON ROAD
GREAT MISSENDEN . BUCKINGHAMSHIRE . HP16 0BD

T: 0800 531 6280 E: mail@triohealthcare.co.uk

TrioOstomyCare.com

trio Pearls™

super absorbent - gelling & odour
control sachets

Essential

...for ileostomates

trusted results in ostomy

MXXXX

trusted results in ostomy

Active Odour Control

Trio Pearls® is not just a gelling agent – it has an organic compound which specifically targets odour.

The breakdown of certain foods in the digestion process produces odorous molecules and in turn gases. Ostomates often fear the loss of control over their bodily function following stoma surgery, and flatus and odour can be a major concern affecting quality of life for ostomates.

Benefits:

- Actively adsorbs odour
- Preserves filter
- Prevents leakage
- Make the pouch more discreet
- Easier drainage/less messy
- Gel does not stain clothes or toilet
- Security over longer periods

Perfect for:

- Overnight use for a good night's sleep
- Long journeys or flights – take it on holiday
- Social occasions such as parties, restaurants, theatre/cinema, etc.
- Active pursuits
- Everyday use

How to Use:

Recommended for ostomates using a drainable pouch, (not suitable for Urostomates).

1. Fasten the drainable end of the pouch/ bag first.
2. Insert the ENTIRE sachet into the pouch from the top - DO NOT empty the content into the pouch (see Diagram 1)

diagram 1

3. Further sachets can be used once you empty by inserting a sachet through the drainable end of the pouch (see Diagram 2)

diagram 2



 trio ostomy care™

Order your FREE sample at: TrioOstomyCare.com