




01 Clean area with water and ensure that the skin is dry. (Attention: The seal will not stick to wet skin)

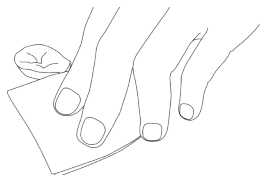

02 Remove by gently rubbing the edge of the seal toward the centre; then using thumb and forefinger carefully but firmly pull from the blister pack. The seal will normally be adhered to the blister.

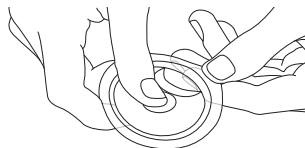

03 Stretch the Trio Siltac® or Trio Silvex® over the stoma and allow it to retract. Once the ring has retracted around the stoma, secure to the body by gently pressing on the seal until secure and comfortable.

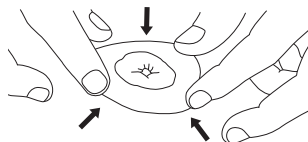

04 Place your ostomy pouching system directly onto the surface of the seal and hold in place with gentle pressure until secure.



Trio Silvex® The advanced, soft silicone convex seal.
Trio Siltac® The advanced, soft silicone ostomy seal.

- Allows the skin to breathe, reducing risk of skin irritation
- Reduced risk of leakage – extends wear time of your ostomy appliance
- Instant adhesion – better seal is formed
- Promotes comfort due to high level of flexibility
- More hygienic than traditional hydrocolloid products as it does not absorb faeces or urine

To find out more or to receive further samples:
please call us on **0800 531 6280**,
email us at **careline@triohealthcare.co.uk**
or visit our website **www.TrioOstomyCare.com**



To watch an instructional video on how to apply your Trio Siltac® or Trio Silvex® seal just visit us at **www.TrioOstomyCare.com**.

Trio Healthcare Ltd., Missenden Abbey, London Road, Great Missenden, Buckinghamshire, HP16 0BD
T: 0800 531 6280 T: 1800 937 139 (IRL) E: careline@triohealthcare.co.uk W: www.TrioOstomyCare.com

Company	Ref	Description	Pack Size
Trio Healthcare Ltd	TR1060	Trio Sillex ® Silicone Flange Extenders	20
	TR1070	Trio Silken ® Silicone Stoma Gel	60g
	TR1020	Trio Siltac ® Silicone Ostomy Seal	20mm-28mm x 30
	TR1028	Trio Siltac ® Silicone Ostomy Seal	28mm-35mm x 30
	TR1035	Trio Siltac ® Silicone Ostomy Seal	35mm-44mm x 30
	TR1020CX	Trio Silvex ® Silicone Convex Seal	20mm-30mm x 10
	TR1030CX	Trio Silvex ® Silicone Convex Seal	30mm-40mm x 10



Take this to your GP to help them add this product to your prescription.