




- 01** Clean area with water and ensure that the skin is dry. (Attention: The flange extender will not stick to wet skin)

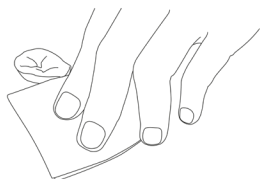


- 02** Remove the clear release liner (Symbol No. 1) as indicated.

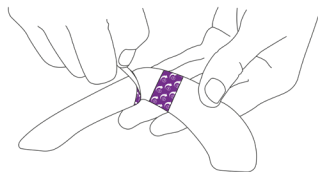

- 03** Position Sillex® where required, overlapping the edge of the flange and your skin.

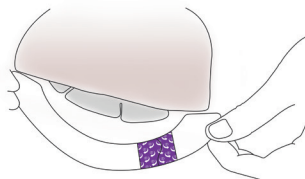

- 04** Remove the paper release liner (Symbol No.2) as indicated and smooth down.

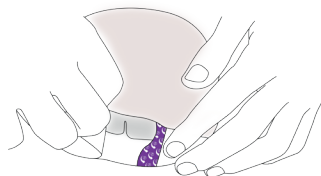

Trio Sillex® The advanced, soft silicone gel flange extenders:

- Responsive Silicone® flange extender
- Ultra thin, like second skin with instant adhesion
- Waterproof and non-absorbent, perfect for swimming and bathing
- Allows skin to breath, reducing risk of skin irritation
- Removes in one piece leaving no residue

To find out more or to receive further samples:

please call us on **0800 531 6280**,

email us at **careline@triohealthcare.co.uk**

or visit our website **www.TrioOstomyCare.com**



To watch an instructional video on how to apply your Trio Sillex® Flange Extender just visit us at **www.TrioOstomyCare.com**.

Trio Healthcare Ltd . Missenden Abbey . London Road . Great Missenden . Buckinghamshire . HP16 0BD

T:0800 531 6280 T:1800 937 139 (ROI) E:careline@triohealthcare.co.uk W:www.TrioOstomyCare.com

Company	Ref	Description	Pack Size
Trio Healthcare Ltd	TR1060	Trio Silex ® Silicone Flange Extenders	20
	TR1070	Trio Silken ® Silicone Stoma Gel	60g
	TR1020	Trio Siltac ® Silicone Ostomy Seal	20mm-28mm x 30
	TR1028	Trio Siltac ® Silicone Ostomy Seal	28mm-35mm x 30
	TR1035	Trio Siltac ® Silicone Ostomy Seal	35mm-44mm x 30
	TR1020CX	Trio Silvex ® Silicone Convex Seal	20mm-30mm x 10
	TR1030CX	Trio Silvex ® Silicone Convex Seal	30mm-40mm x 10
	TR201	Trio Elite® (Spray) Adhesive Remover	50ml
	TR202	Trio Elite® (Wipes) Adhesive Remover	30 Wipes
	TR203	Trio Elisse® (Wipes) Skin Barrier	30 Wipes
	TR204	Trio Elisse® (Spray) Skin Barrier	50ml
	TR205	Trio Pearls® Gelling Sachets	100 Sachets



Take this to your GP to help them add this product to your prescription.