



- 01** To open the tube, unscrew the cap. Puncture the foil with the reverse side of the cap.



- 02** Clean area with water and ensure that the skin is dry.

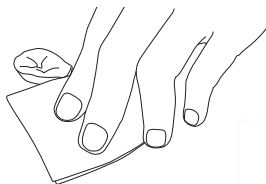

- 03** Squeeze the gel to fill dips and creases where required. For best results do not spread the gel.

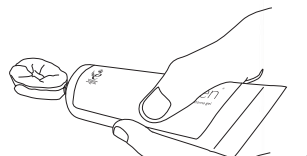

- 04** Apply your ostomy wafer or pouching system directly onto the gel and allow it to spread naturally. The gel will set underneath the wafer/pouching system during normal wear time.



Trio Silken® The advanced, soft silicone stoma gel:

- Fills skin folds, creases and uneven areas around the stoma
- Will not sting on application
- Allows the skin to breathe
- One piece removal - leaves no residue*
- Remains soft and flexible to ensure comfort

To find out more or to receive further samples:

please call us on **0800 531 6280**,

email us at **careline@triohealthcare.co.uk**

or visit our website **www.TrioOstomyCare.com**



*when fully cured

To watch an instructional video on how to apply your Trio Silken® stoma gel just visit us at **www.TrioOstomyCare.com**.

Trio Healthcare Ltd., Missenden Abbey, London Road, Great Missenden, Buckinghamshire, HP16 0BD

T: 0800 531 6280 T: 1800 937 139 (ROI) E: careline@triohealthcare.co.uk W: www.TrioOstomyCare.com

Company	Ref	Description	Pack Size
Trio Healthcare Ltd	TR1060	Trio Sillex ® Silicone Flange Extenders	20
	TR1070	Trio Silken ® Silicone Stoma Gel	60g
	TR1020	Trio Siltac ® Silicone Ostomy Seal	20mm-28mm x 30
	TR1028	Trio Siltac ® Silicone Ostomy Seal	28mm-35mm x 30
	TR1035	Trio Siltac ® Silicone Ostomy Seal	35mm-44mm x 30
	TR1020CX	Trio Silvex ® Silicone Convex Seal	20mm-30mm x 10
	TR1030CX	Trio Silvex ® Silicone Convex Seal	30mm-40mm x 10



Take this to your GP to help them add this product to your prescription.