



ready to get Genii™ Ostomy Seals added to your prescription?

Our flat and convex seals are part of a complete range of Genii™ skincare solutions available on prescription and stocked by all dispensing suppliers and ostomy delivery companies. Your Trio Advisor will keep in contact to help with anything you need.

Your stoma may change shape and size but our seals will flex and move with your stoma. Simply choose the best size for you by selecting your stoma size and choosing small, medium or large.

Genii™ Ostomy Seals

Code	Size	Stoma size	Pack size
TR1020	Small	20–35mm	30
TR1028	Medium	35–50mm	30
TR1035	Large	50–80mm	30

Genii™ Convex Ostomy Seals

Code	Size	Stoma size	Pack size
TR1020CX	Small	20–35mm	10
TR1030CX	Large	30–50mm	10

Genii™ Flange Extenders

Code	Size	Stoma size	Pack size
TR1060	—	All	20



Need extra security?

You may have a hernia, do a lot of physical activity or want peace of mind at night. Genii™ Flange Extenders are designed to go around the flange to provide a soft, flexible, invisible border which conforms to your body shape to provide extra security. Get in touch today if you'd like more information or to order your free sample




trioostomycare.com

0800 531 6280

careline@triohealthcare.co.uk



Genii™ Ostomy Seals genius for preventing leaks

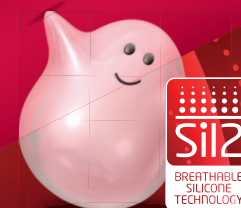

Trio Healthcare Ltd. Registered in England. No: 05776276. Registered office: Units 2A-2B, Sandylands Business Centre, Carleton New Road, Skipton, North Yorkshire, BD23 2AA. United Kingdom. SiL2 Breathable Silicone Technology® and Trio Genii™ are trademarks of Trio Healthcare Ltd ©2021 Trio Healthcare Ltd

M0136



the first silicone ostomy seal that provides a perfect fit.

Genii™ Ostomy Seals are available in flat and convex to suit all stoma types



Snug, customised fit
Hugs the stoma, eliminating gaps and leaks



Comfortable and secure
Responds to body movement



Quick and easy to apply
Simply stretch around the stoma without warming or shaping



Protects delicate skin around your stoma
Protection from stool and urine, allowing it to heal naturally and remain healthy




Longer wear time
Unique Sil2 Breathable Silicone Technology® doesn't absorb bodily fluids so it won't go gooey, mushy or breakdown

ALSO AVAILABLE

Genii™ Convex Ostomy Seals
Ideal for flat or inverted stomas or uneven skin.



why is Sil2 Breathable Silicone Technology® great for skin?

Sil2 Breathable Silicone Technology® is a unique silicone adhesive blend that feels and breathes just like normal skin. It sticks brilliantly but doesn't absorb moisture. So, it won't swell, go mushy, create a gooey mess, or cause pain when you remove it. Which means you can wear it for longer without worrying about leaks.

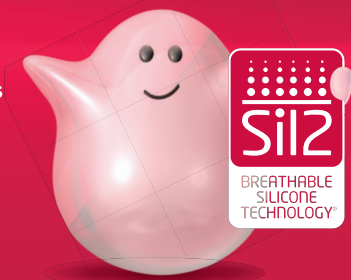


how to use a Genii™ Ostomy Seal

1. Clean the area around the stoma, it's important to make sure the skin is dry as **the seal won't stick to wet skin.**
2. Using your thumb and forefinger, carefully pull the seal from its pack. It's normal for the seal to be fixed to the blister pack, but it's strong enough for you to pull and stretch it free without damage.
3. Take the edge of the seal in both hands and stretch it around the stoma. You can then allow the seal to retract around the stoma to form a comfortable seal.
4. The Genii™ Ostomy Seal stretches to fit around the stoma. So there's no need to mould it into shape. Its gentle elasticity will hug the stoma to form the seal.
5. Once it has sealed around the stoma, gently press the Genii™ Ostomy Seal onto your skin. Place the ostomy bag directly onto the seal as normal and hold it in place until it feels secure.

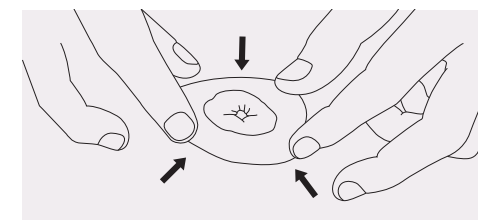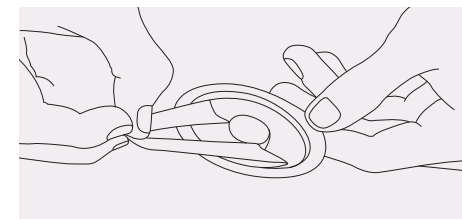

3. Take the edge of the seal in both hands and stretch it around the stoma. You can then allow the seal to retract around the stoma to form a comfortable seal.

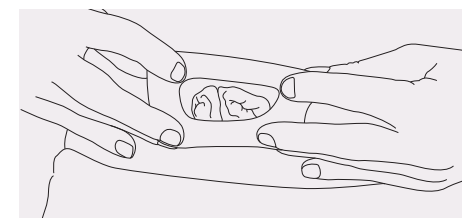

Convex seals are versatile and can be applied in two ways, either convex side up or convex side down, so that gentle pressure can be applied directly around your stoma to create a secure fit, whichever works best for you.