



Trio Elite® Sting Free Adhesive Remover

say goodbye
to painful
removals

trio Elite®



does your adhesive remover sting when you spray it onto your skin?

Trio Elite® gets to work on tough adhesives for fast and pain-free removal.



Quick and easy to apply for fast and pain-free stoma bag removal



Allows skin to breathe
Doesn't leave any residue or wetness on the skin



Kind to skin
Our unique silicone formula is sting free and kinder to skin



Prevents skin damage
by reducing skin stripping caused by repeated bag changes



Dries in seconds
Fast drying



360° directional spray
Sprays at any angle

Trio Elite® Silicone Technology

Trio Elite® Sting Free Adhesive Remover has a hypoallergenic 100% silicone formula that won't irritate the skin and quickly gets to work on tough adhesives. The directional spray action ensures Trio Elite® is applied only where you want it, without wastage.

how to use Trio Elite®

Trio Elite® uses bag in can technology – a recyclable and sustainable packaging system. Free from pollutants used in traditional aerosols, this packaging approach ensures 100% of the product is used.



1. Hold the Trio Elite® can approximately 10cm (4-5 inches) away from the skin and spray at the top of the adhesive flange.
2. Allow the adhesive remover to take effect. You should be able to feel the skin separate from the adhesive.
3. Peel away adhesive flange from the skin and remove bag, continue to spray the Trio Elite® to aid removal if required.
4. If you are left with any residue on your skin from your adhesive flange, spray Trio Elite® and gently wipe to remove. Alternatively, you can use Trio Elite® wipes.



Also available in wipes

Handy size, individually wrapped for when you are travelling or out and about.

Trio Elite® is available on prescription and stocked by all dispensing suppliers and ostomy delivery companies.

We hope you've enjoyed using Trio Elite®. Please don't hesitate to get in touch with a Trio advisor should you like any further information or need additional advice.

Trio Elite® spray


Code	Pack size
TR201	50ml spray

Trio Elite® wipes

Code	Pack size
TR202	30 wipes



 trioostomycare.com

 0800 531 6280

 careline@triohealthcare.co.uk

