



**Trio Pearls® Gelling and
Odour Control Sachets
say hello to
confidence**

trio Pearls®



are you worried about leaking,
odour or output 'sloshing' around
in your stoma bag?

**Trio Pearls® is a super absorbent
gelling and odour control sachet.**



Prevents leakage
by solidifying liquid output



Controls odours
Actively adsorbs odorous molecules
and gases



Colourless
Doesn't stain clothes or toilet

**Trio Pearls® mean less
stress and less mess**



Trio Pearls® NanoActive Technology



Trio Pearls® combine super absorbent gelling crystals and nano-engineered ingredients that actively absorb odorous organic molecules. Providing greater security when you don't have easy access to empty or change your stoma bag.

Giving confidence in all aspects of your life

- Use overnight for a good night's sleep
- Take it on holidays, great for long journeys
- Use every day, giving you more control over your stoma and your life
- Keep Trio Pearls® handy for peace of mind at social events

how to use Trio Pearls®

1. Simply pop the Trio Pearls® sachet inside your new stoma bag. You don't need to empty the sachet.
2. Empty output as normal into the toilet or if you are using a closed bag simply dispose as normal.
3. You can continue to add sachets after you have emptied your output by inserting another sachet through the drainable outlet.

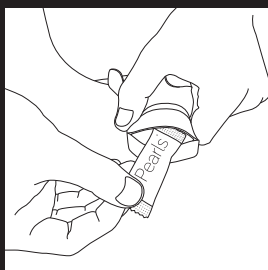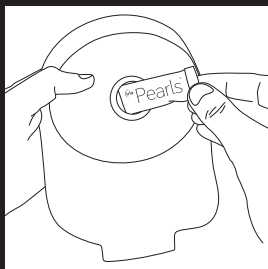

For ileostomy or liquid stool output only.

Trio Pearls® is available on prescription and stocked by all dispensing suppliers and ostomy delivery companies.

We hope you've enjoyed using Trio Pearls®. Please don't hesitate to get in touch with a Trio advisor should you like any further information or need additional advice.

Trio Pearls®

Code


Pack size

TR205

100 sachets



 trioostomycare.com

 0800 531 6280

 careline@triohealthcare.co.uk

