



Trio Silken® Silicone Stoma Gel

**say goodbye
to leaks**

trio Silken®



is the skin surrounding your stoma uneven and causing leaks?

Trio Silken® is a sting free silicone gel that creates an even surface for optimum stoma bag application.



Allows skin to breathe

Reduces skin damage and keeps skin healthy



Flexible and secure

Responds to body movement, filling the gaps caused by skin folds, dips, creases and uneven skin



Quick and easy

Removes in one piece and doesn't leave residue on the skin



Protects delicate skin around your stoma


Prevents output getting into contact with the skin

Trio Silken[®] silicone stoma gel made with Sil2 Breathable Silicone Technology[®]



how to use Trio Silken[®]

1. Prepare the area around your stoma ensuring that the skin is clean and dry.
2. To open, unscrew the cap and puncture the foil with the reverse side of the cap.
3. Gently squeeze the gel to fill dips and creases as required. Only a small amount of Trio Silken[®] is needed.
4. There's no need to spread the gel or wait until it's dry, simply apply your stoma bag as normal, placing it directly onto the moist gel. Trio Silken[®] will solidify slowly, remaining soft until the creases and gaps in your skin have been filled.

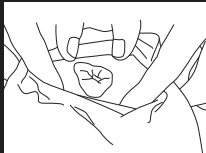

Trio Silken® is available on prescription and stocked by all dispensing suppliers and ostomy delivery companies.

We hope you've enjoyed using Trio Silken®. Please don't hesitate to get in touch with a Trio advisor should you like any further information or need additional advice.

Trio Silken®

Code

Pack size

TR1070

60g tube



trioostomycare.com



0800 531 6280



careline@triohealthcare.co.uk

