



GeniiTM

the stoma bag that
sticks, seals and feels
like it's not even there

trio
GeniiTM



Genii™ sticks, seals and feels like it's not even there.

Thanks for trying Genii.™ It's the first ever stoma bag in the world that uses our unique Sil2 Breathable Silicone Technology® in the flange. Which means it sticks with a unique silicone adhesive that seals perfectly to protect skin against output and moves with your body so it feels like it's not even there.

With a lightweight, water-resistant material in a choice of three skin tone colours, we know you're going to love it!



Carbon filter



Pull tab / easy opening



Discreet fold / tuck



Water-resistant



**Sticks with a unique
silicone adhesive**

which is naturally breathable

Seals perfectly to protect skin

with no pain or damage on removal

Feels like it's not even there

Soft, flexible and conformable

Water-resistant material

Leaves the bag dry to the touch. Ideal for being active, swimming, showering and bathing

Skin tone colours

'Camouflages' against the skin - available in a choice of light, medium and dark

Soft material cover

Lightweight, quiet material, stays fresh for optimum, long-lasting comfort

Improved airflow

High performance carbon filter manages air in the bag to reduce ballooning, pancaking and odour

Easy viewing

Split design bag cover to help view contents

Easy to empty and clean

Nifty pull tab allows easy opening, draining, and cleaning

Genii™ clinical study.¹

Our clinical assessment of Genii™ has been a roaring success. We were thrilled to see so many ostomates noticing improvement to their skin after just one week, plus all of this great feedback!



Genii™ scores 9/10 for stickability...

“

The soft adhesion disc fitted very well over my hernia and felt very comfortable.

”



93% said they would recommend Genii™...

“

It took me a little while to get used to it, but once I did, I wouldn't go back. Just give it time to get used to it – it will be worth it.

”



Genii™ scores 9.2/10 for comfort...

“

The bag is more comfortable, easy to wear, apply and remove. There has also been less irritation to my skin since using the bag.

”

“

Skin has improved, I'm less self-conscious and not aware of pouch.


”

1. Trio data on file 2021.

tips to get a great fit.




You'll find a detailed instructions for use leaflet inside this pack. Please refer to these instructions before use. As a general guide:

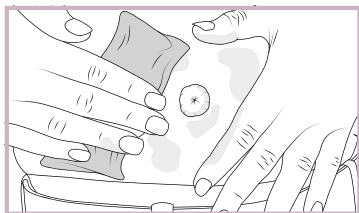


Step 1

Gently clean around the stoma with lukewarm water, soft tissues, cotton wool, or skin cleanser wipes. Make sure the skin is completely dry.

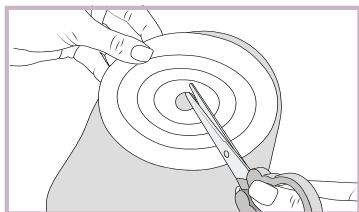


Step 2

Cut the silicone flange to size. A stoma size guide is included to help you with this. The texture of the silicone flange will feel a little bit different compared to your current product. It feels easier to cut due to silicone being a softer and more flexible adhesive.

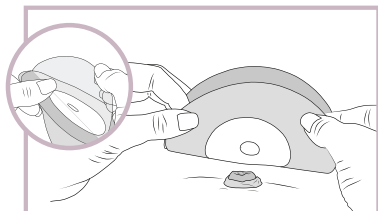


Step 3

Remove the protective cover from the flange and fit the bag over the stoma. Smooth the edges to make sure there are no creases.

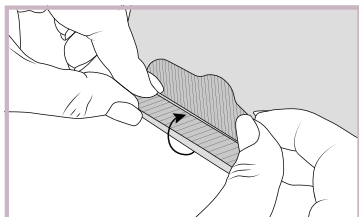

Step 4

Fold the drainable tabs upwards and secure. Once secure, fold the outlet one last time and tuck underneath the bag's cover.

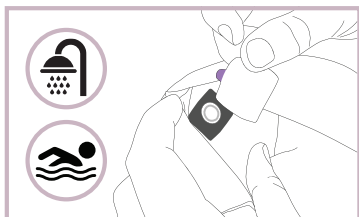

Step 5

If showering, bathing or swimming please place one of the stickers provided over the bag filter.

people who've tried Genii™ haven't looked back.

We're confident our new Genii™ stoma bags are going to be life changing. Anne, Alfred and Scott were amongst the first to try it.

Genii™ sticks...

“

What amazed me was the new silicone adhesive stuck much more firmly than anything I've used before. I had no leaks or pancaking. The silicone flange makes all the difference.

Alfred

”

Genii™ seals...

“

I found the silicone product actually clung to my skin. There was no peeling of the flange even after days. It gave me great confidence wearing it and going out to exercise.

Scott

”

Genii™ feels...

“

Once it was on, you couldn't feel it and that's massive because you can forget it's there. It worked incredibly well, there were no gaps and no wrinkling. I think it's a real gamechanger.

Anne

”

choose from a range of Genii™ accessories.

Trio also has a range of essential accessories containing Sil2 Breathable Silicone Technology®. They have all been designed to help you combat common problems and promote healthy stoma skin.

Genii™ Ostomy Seals

Genii™ Ostomy Seals protect the delicate skin around the stoma from stool and urine, allowing it to heal naturally and remain healthy. Quick and easy to apply, they simply stretch to provide a custom fit, without the need for warming or cutting.



Genii™ Convex Ostomy Seals

Dips, gaps or uneven skin around your stoma can cause leaks. Genii™ Convex Ostomy Seals are ideal for flat and inverted stomas. They fit smoothly to fill any gaps and apply gentle pressure, if needed, around the stoma.



Genii™ Flange Extenders

If you have a hernia, do lots of physical activity or want peace of mind at night, Genii™ Flange Extenders offer extra security. They're designed to go around the flange to form a soft, flexible, invisible border which conforms to your body shape.

Genii™ Ostomy Seals

| Code | Size | Stoma size | Pack size |
|--------|--------|------------|-----------|
| TR1020 | Small | 20–35mm | 30 |
| TR1028 | Medium | 35–50mm | 30 |
| TR1035 | Large | 50–80mm | 30 |

Genii™ Convex Ostomy Seals

| Code | Size | Stoma size | Pack size |
|----------|-------|------------|-----------|
| TR1020CX | Small | 20–35mm | 10 |
| TR1030CX | Large | 30–50mm | 10 |

Genii™ Flange Extenders

| Code | Size | Stoma size | Pack size |
|--------|------|------------|-----------|
| TR1060 | — | All | 20 |

ready to switch your prescription to Trio Genii™?

Call us on 0800 531 6280 (UK)

Email us at careline@triohealthcare.co.uk

Visit trioostomycare.com

Light

| Pip Codes | Code | Size | Flange | Bag size | Filter | Pack Size |
|-----------------------|--------|---------|------------|----------|--------|-----------|
| Drainable bags | | | | | | |
| 416-5395 | TR324N | 15-60mm | cut to fit | Mini | ✓ | 30 |
| 416-5361 | TR325N | 15-60mm | cut to fit | Midi | ✓ | 30 |
| 416-5338 | TR336N | 15-60mm | cut to fit | Maxi | ✓ | 30 |


Medium

| Pip Codes | Code | Size | Flange | Bag size | Filter | Pack Size |
|-----------------------|--------|---------|------------|----------|--------|-----------|
| Drainable bags | | | | | | |
| 416-5387 | TR324T | 15-60mm | cut to fit | Mini | ✓ | 30 |
| 416-5353 | TR325T | 15-60mm | cut to fit | Midi | ✓ | 30 |
| 416-5320 | TR336T | 15-60mm | cut to fit | Maxi | ✓ | 30 |

Dark

| Pip Codes | Code | Size | Flange | Bag size | Filter | Pack Size |
|-----------------------|--------|---------|------------|----------|--------|-----------|
| Drainable bags | | | | | | |
| 416-5379 | TR324C | 15-60mm | cut to fit | Mini | ✓ | 30 |
| 416-5346 | TR325C | 15-60mm | cut to fit | Midi | ✓ | 30 |
| 416-5312 | TR336C | 15-60mm | cut to fit | Maxi | ✓ | 30 |

Bags with split cover

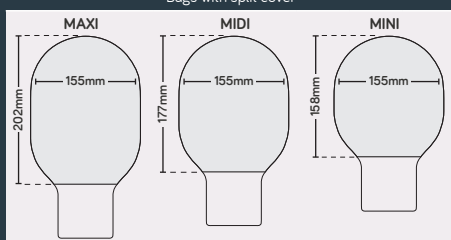

All sizes and shades are available on prescription and stocked by all dispensing suppliers and ostomy delivery companies.

Your Trio Advisor will keep in contact to help with anything you need.

We hope you've enjoyed trying new Trio Genii™.

Tell us about your experience by completing the feedback form in this pack and return your completed form in the pre-paid envelope. Alternatively you can fill in the form on-line by scanning the QR code, or visit www.surveymonkey.co.uk/r/genii2022



trio
Genii
Genius for skin.



trioostomycare.com 0800 531 6280 careline@triohealthcare.co.uk



Trio Healthcare Ltd. Registered in England. No: 05776276. Registered office: Units 2A-2B, Sandylands Business Centre, Carleton New Road, Skipton, North Yorkshire, BD23 2AA, United Kingdom. Sil2 Breathable Silicone Technology® and Trio Genii™ are trademarks of Trio Healthcare Ltd. ©2022 Trio Healthcare Ltd.

M0181