

# How to apply



Hello, my name is Genii™ and I'm made from silicone.  
I'm not like any stoma bag you've tried before.

I'm made from a unique Sil2 Breathable Silicone Technology® which sticks firmly to the skin giving you a secure and comfortable fit from the very first application. Silicone is super sticky and super stretchy, we advise you to take a little extra care when removing the protective cover and recommend you prepare the Genii™ stoma bag before you remove your existing bag.

1.



Cut a hole in the silicone flange to the size of your stoma. The hole needs to be snug on application, use the size guide that's inside the box to help you.

2.



Remove the 2 layered protective cover from the silicone. Ensure you remove both parts - see image A.



3.



Using your thumb peel away the white silicone from the plastic protective cover.

4.



Once the silicone adhesive flange has come away from the protective cover, gradually work around the edges using your thumb and fingers.

5.



Continue to peel away the edges of the protective cover until you have separated approx 50% around the silicone adhesive flange.

6.



Pull the release liner from left to right and remove completely. Don't worry if the silicone stretches as you do this, this is completely normal.

7.



The silicone adhesive flange will return to its original shape once the protective cover has been removed.

8.



Remove your existing stoma bag and ensure your skin around your stoma is clean and dry. Apply adhesive remover or skin barrier as required. **Please ensure your skin is dry as silicone will not stick to wet skin.**

**Don't worry if your bag feels a little different when you put it on, it's designed to feel like a second skin - you may even forget it's even there! If you do experience any problems our team is always there to help.**